



Wrist Brace

Fitting / Care Instructions

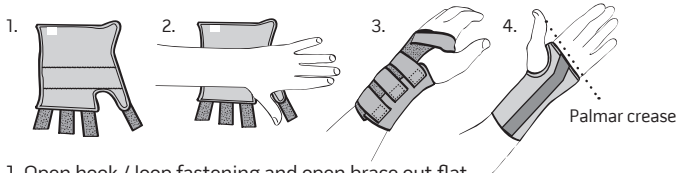

1. Open hook / loop fastening and open brace out flat.
2. Position hand in brace as shown
3. Close hook / loop fastening and re-adjust for comfort as required.
4. Ensure end of brace is not above palmar crease.

Care Instructions

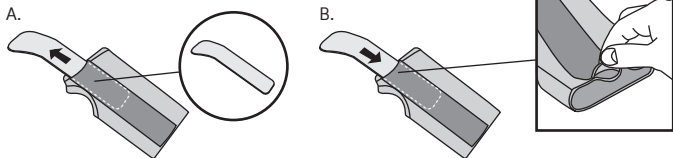

1. Remove metal splint (See diagram A)
2. Close over all hook/loop straps.
3. Hand wash brace in cool water.
4. Rinse.
5. Dry flat.
6. Do not spin or tumble dry.

Important:

Before using brace again, replace metal splint ensuring splint is correct way round and splint is tucked under edge / 'lip' of pocket. (See diagram B)



SCAN QR CODE TO WATCH

WRIST BRACE FITTING VIDEO

www.youtube.com/promedicsorthopaedic



Telephone: +44 (0) 1475 746 400
Sales Fax: +44 (0) 1475 746 410
General Fax: +44 (0) 1475 746 411
E-mail: enquiries@promedics.co.uk

Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate
Gareloch Road, Port Glasgow
Scotland PA14 5XH