


COT - Carpal Brace

EN	WRIST/HAND ORTHOSIS
DE	HANDGELENK-/HANDORTHESE
FR	ORTHÈSE DU POIGNET/DE LA MAIN
ES	ORTESIS DE MUÑECA/MANO
IT	ORTESI PER POLSO/MANO
NL	POLS-HANDORTHESE
DK	HÅNDLEDS-/HÅNDORTOSE
FI	RANNE-KÄMMENORTOOSI
SE	HANDLEDS- OCH HANDORTOS
NO	ORTOSE, HÅNDLEDD/HÅND
PL	ORTEZA NADGARSTKA/DŁONI
GR	ΝΑΡΘΗΚΑΣ ΚΑΡΠΟΥ/ΧΕΙΡΙΟΥ

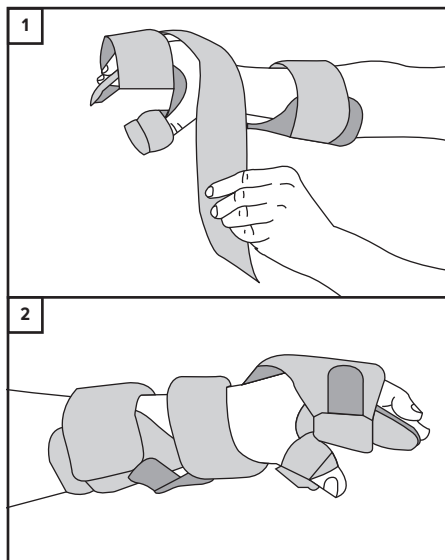

Product Description

Malleable aluminium frame covered with perforated breathable material. The orthosis stabilises the wrist, hand and fingers and can easily be adapted to the size of the wrist and thumb of the patient. Its anatomical shape provides comfortable and effective therapy.

Usage Information (See diagram)

1. Position the wrist at the desired position on the brace.
2. While holding the wrist in place, secure the forearm and wrist straps.
3. Position the fingers on the finger platform and secure the hand strap to maintain finger positioning in the device.
4. Position and secure the thumb with the thumb strap.

Indications for Use

- Carpal Tunnel Syndrome.
- Mild sprains.
- Mild strains.
- Post-cast removal.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Family Composition

PU Foam, Aluminium, Polyester, Nylon.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

 **MEDICAL DEVICE**

 **SINGLE PATIENT - MULTIPLE USE**

 **CAUTION**



 Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

   **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands