MFS - Mallet Finger Splint

© 2024 Promedics Orthopaedics Ltd. All Rights Reserved.
MFSIFUE revised 6/24

EN HAND/FINGER SPLINT, REUSABLE

DE HAND-/FINGERSCHIENE, WIEDERVERWENDBAR
FR GOUTTIÈRE POUR LA MAIN/LES DOIGTS, RÉUTILISABLE
ES FÉRULA PARA DEDO/MANO, REUTILIZABLE
IT SPLINT PER MANO/DITO, RIUTILIZZABILE

NL HAND-/VINGERSPALK, HERBRUIKBAAR

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ENGLISH

Product Description

A finger orthosis for immobilisation of the distal interphalangeal (DIP) joint of the digits in neutral.

Usage Information (See diagram)

- Decide on size required.
- Slide orthosis onto the required digit so that the nail can be see through the opening as in image 3.
- Secure in place by desired means such as self-adhesive straps. Take care not to over tighten straps, re-adjust for comfort as required while ensuring orthosis is in correct position.

Indications for Use

- For digit injuries / conditions that require immobilisation / support of the distal interphalangeal (DIP) joint in neutral such as:
 - Mallet injuries.
 - Fractures.
 - Joint instability.
 - Nail bed injuries.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.

Washing and Care Instructions

- Wash in warm soapy water and rinse.
- Ensure the product is dry before re-use.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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