## AC - Ankle Contracture

EN ANKLE/FOOT ORTHOSIS

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ES ORTESIS DE TOBILLO/PIEDE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORTHESE

DK ANKEL-/FODORTOSE

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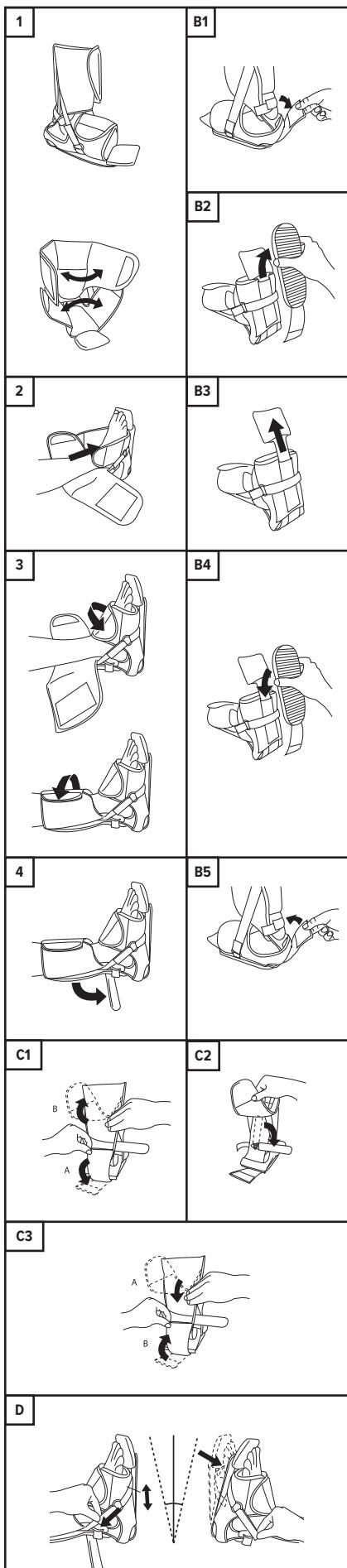
SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

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### ENGLISH

#### Product Description

A quality AFO constructed from lightweight polycarbonate that provides heel suspension to assist in the treatment and prevention of pressure sores.

#### Usage Information (See diagram)

1. Open the hook/loop fastenings on liner.
2. Position foot ensuring heel is positioned in recess and toe rest is correct length. (See steps B1 - B5).
3. Close fastenings on liners ensuring they are firm but not too tight.
4. Position anti-rotation bar if required (See steps C1 - C3).
- B1. To adjust the length / toe rest. Unfasten the hook/loop.
- B2. Slide the sole off.
- B3. Pull the toe rest (there will be some resistance from the inner hook fixation). Extend length to accommodate larger foot.
- B4. Slide sole back into slot.
- B5. Re-fasten the hook/loop flap at heel. Always secure the fastening.
- C1. To position the anti-rotation bar, unfasten the hook/loop flap at heel (A). Then open liner (B) as shown.
- C2. Rotate the bar to desired position.
- C3. Close liner as shown. Always ensure secure fastening.
- D. Ensure dorsiflexion assist straps are located to front of foot for best leverage. The strap underneath the foot can be easily repositioned. Tighten straps evenly on both sides.

#### Indications for Use

- Pressure sores.
- Foot drop/non-fixed plantar flexion.
- Prevention of lateral/medial hip rotation.

#### Contraindications

- Any known allergy and/or hypersensitivity to any material listed in product composition.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the brace is not being worn or when being washed.

#### Washing and Care Instructions

1. Close over all hook/loop straps & remove liner.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

#### Product Family Composition

ABS, Aluminium, Cotton, Nylon, Polycarbonate, POM, PU Foam.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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