AFS - Armchair Finger Splint

EN HAND/FINGER SPLINT, REUSABLE

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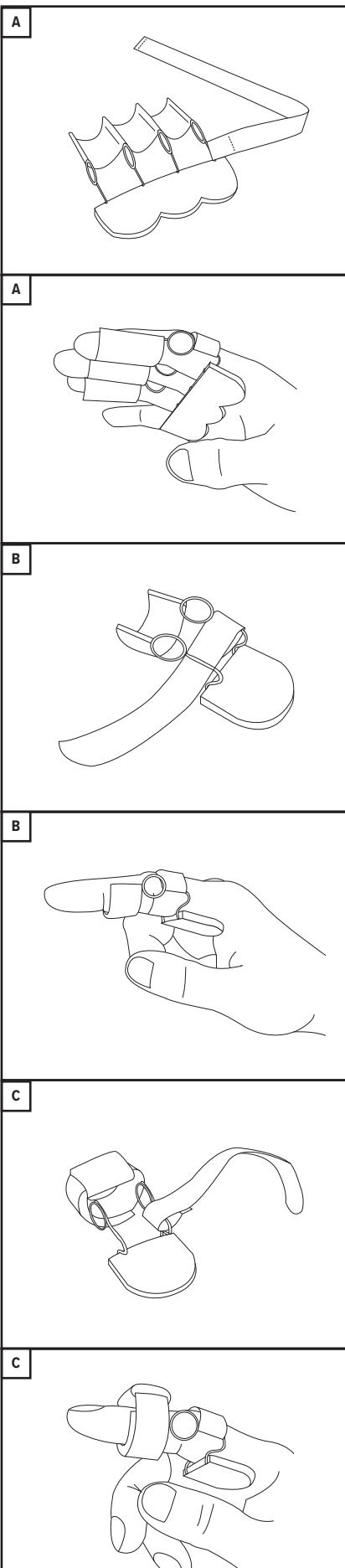
ENGLISH

Product Description

A dynamic splint with coil design for PIP and DIP joint flexion contractures and Boutonniere's deformity.

Usage Information (See diagram)

1. Open all hook / loop fastenings.
2. Position the finger in brace as shown.
3. Close hook / loop fastening.
4. Re-adjust for comfort.
5. Take care not to over tighten orthosis, re-adjust for comfort as required while ensuring brace is in correct position.

Indications for Use

Provides finger support / extension for an array of injuries, conditions, pre and post operations such as:

- PIP and DIP joint flexion contractures.
- Extensor tendon injuries / conditions (Boutonniere's deformity).
- Soft tissue conditions and injuries.
- Maintains assisted extension at the PIP joint while allowing active flexion.

It is recommended that individuals seek guidance from a medical practitioner regarding the most appropriate brace and wearing schedule.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Polyamide, PU, Polyethylene, Polyester, Nylon, Steel.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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