
AS - Arm Sling

EN	ARM IMMOBILIZER
DE	ARM-IMMOBILISATIONSVERBAND
FR	IMMOBILISATEUR DE BRAS
ES	INMOVILIZADOR DE BRAZO
IT	IMMOBILIZZATORE PER BRACCIO
NL	ARMMIMMOBILISATOR
DK	ARMSTØTTE
FI	KÄSIVARREN IMMOBILISAATIOTUKI
SE	ARMSKENA
NO	IMMOBILISATOR, ARM
PL	WYRÓB DO UNIERUCHAMIANIA RĘKI
GR	AKINHTOPOIHTHΣ BPXHIONA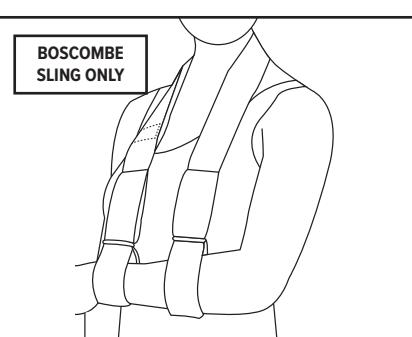

ENGLISH

Product Description

A universal collar and cuff sling to provide support / immobilisation to the upper limb. Features an adjustable buckle for enhanced comfort.

Usage Information (See diagram)

- Fasten the hook fastening on the end of the sling to create an enclosed ring.
- Place this large ring over your head.
- Slide your arm through the smaller / second ring which is already formed at one end by a toggle.
- The hook fastening can be attached anywhere on the sling making this orthosis universal and adjustable so that the right position for the arm can be achieved.

Take care not to over tighten straps, re-adjust for comfort as required while ensuring orthosis is in correct position.

Indications for Use

- Can be used for an array of injuries and conditions where the restriction of shoulder and/or upper arm and / or elbow and / or wrist movement is required such as:

- Dislocation.
- Strains and sprains.
- Pre and post operatively.
- Rehab stages (post fractures or soft tissue injuries).
- Brachial plexus or peripheral nerve.
- Other injuries.

Contraindications

- Do not use open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash sling in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Nylon, PU, POM, Polyester.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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