AWS - Wedge & Strap Kit

EN ANKLE/FOOT ORTHOSIS

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ES ORTESIS DE TOBILLO/PIE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORTHESE

DK ANKEL-/FODORTOSE

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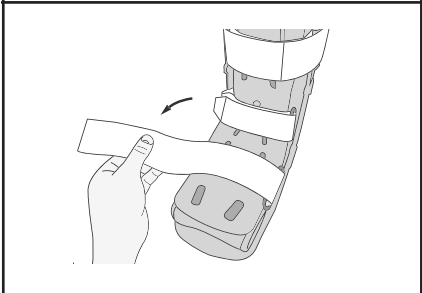
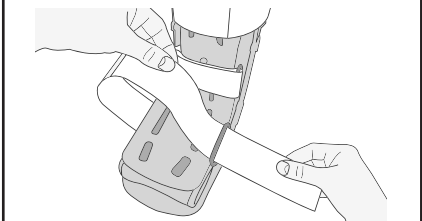
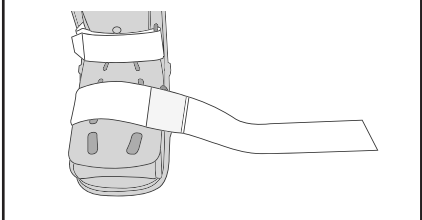
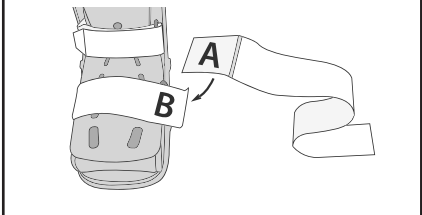
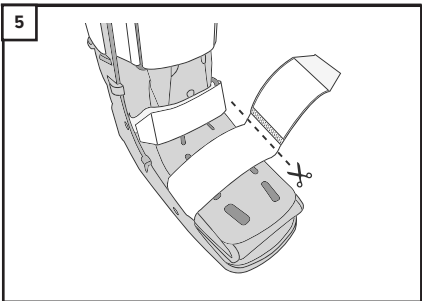
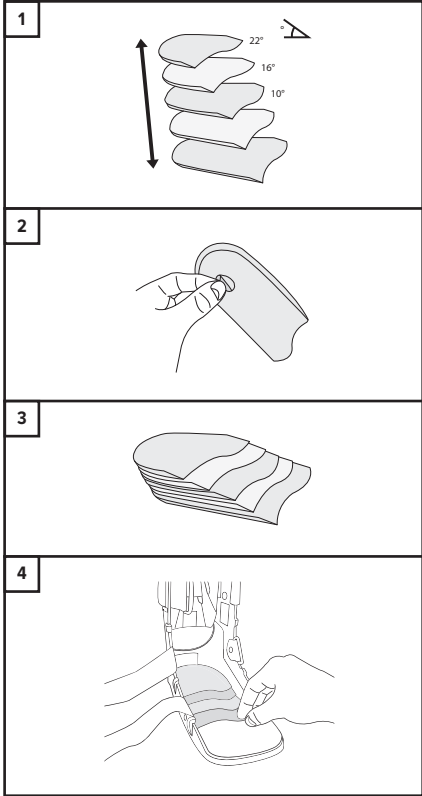
SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

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ENGLISH

Product Description

Achilles Wedge is designed to comfortably place the foot in plantar flexion to offload the Achilles tendon during rehab. The Achilles Wedge provides customized height adjustments and can be used with Jura Walkers.

Usage Information (See diagram)

- The wedges can be placed on top of each other to achieve the desired heel rise. 3 wedges = 22° of heel raise, 2 wedges = 16° of heel raise, 1 wedge = 10° of heel raise.
- Fix wedges together using adhesive backing.
- Wedges should something similar to diagram.
- Affix wedges into heel of walker using the adhesive backing.
- If a strap extension kit has been purchased please attach extension straps as shown.

Indications for Use

Achilles Wedge is designed to comfortably place the foot in plantar flexion to offload the Achilles tendon during rehab. The Achilles Wedge provides customised height adjustments.

Contraindications

- Do not use over open wounds.
- Any known allergy and/ or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
 - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Hand wash wedges in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.

Product Family Composition

EVA, Nylon.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.