
CC - Cervical Collar

EN	CERVICAL COLLAR
DE	ZERVIKALORTHESE
FR	COLLIER CERVICAL
ES	CUELLO CERVICAL
IT	COLLARE CERVICALE
NL	NEKBRACE
DK	HALSHVIRVELKRAVE
FI	KAULATUKI
SE	NACKKRAGE
NO	CERVIKAL KRAGE
PL	SZYJNY KOLNIERZ ORTOPEDYCZNY
GR	ΑΥΧΕΝΙΚΟ ΚΟΛΛΑΡΟ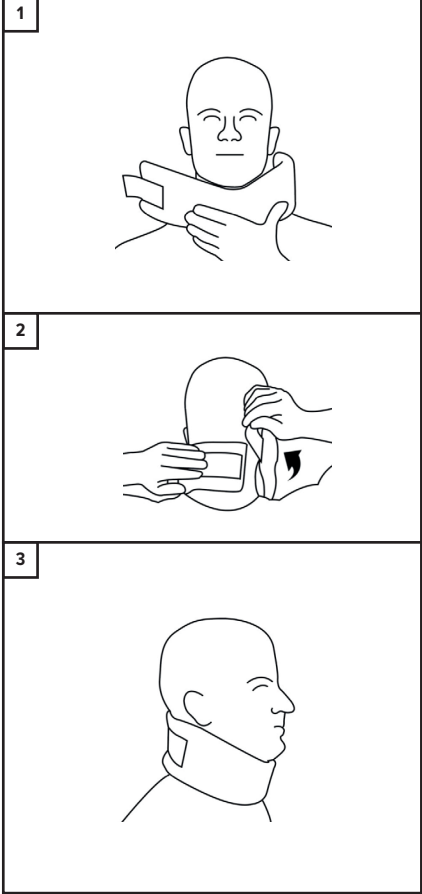

ENGLISH

Product Description

Soft collar for light support of the head / neck.

Usage Information (See diagram)

- Open the collar and position centrally under the chin.
- Wrap the collar around the neck and affix the hook/loop strap.
- Ensure the strap is secure and the collar is fitted firmly, but not too tightly.

Indications for Use

- Light support following minor injury.
- Whiplash.
- Neck muscle spasm.

Contraindications

- Do not use directly on broken skin.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Cotton, Lycra, Nylon, Polyamide, Polyester, Polyether, Polyurethane.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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Promedics Orthopaedics Ltd.