## EC - Epicondylitis Clasp

### EN ELBOW ORTHOSIS

### DE ELLENBOGENORTHESE

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### IT ORTESI PER GOMITO

### NL ELLEBOOGORTHESE

### DK ALBUEORTOSE

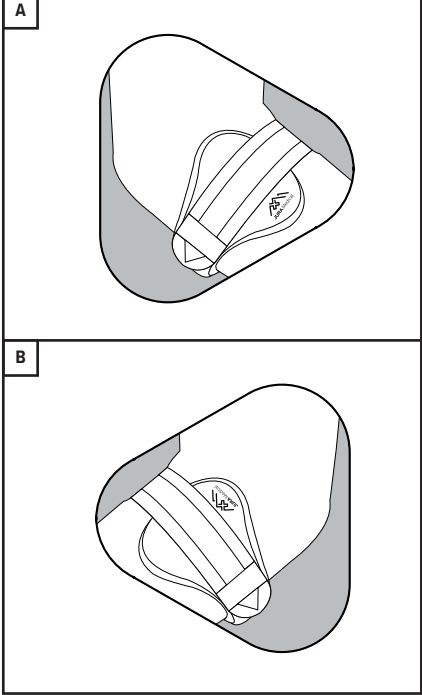
### FI KYYNÄRNIVELORTOOSI

### SE ARMBÅGSortos

### NO ORTOSE, ALBUE

### PL ORTEZA ŁOKCIA

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#### ENGLISH

#### Product Description

Designed to reduce the pain and discomfort associated with elbow tendinopathies.

#### Usage Information (See diagram)

- Open the elastic strap.
- Apply the Jura Clasp to the forearm area as shown with the larger pad on the outside of the arm and the clasp contouring the back of the forearm.
- Whilst applying minimal force, pull the elastic strap across the arm and position the white arrow on the scale between 1-10, to the desired level of compression, ensure the hook/loop fastening is secured.
- Make a record of the position to aid repeated accurate fitting of the brace.
- The brace can fit left and right and when fitted to the left arm the Jura Medical logo appears upside down.
- A. Shows brace fitted to right arm, B. shows brace fitted to left arm.

Measure wrist circumference to determine size selection.

#### Indications for Use

- Tendinopathies.
- Epicondylitis.
- Tennis Elbow.
- Golfer's elbow.

#### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

- Remove strap.
- Hand wash strap and clasp.
- Air dry.
- Replace strap on clasp.

#### Product Family Composition

Nylon, Polycarbonate, Polyester, Polyethylene, Silicone.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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