


## ECS - Epicondylitis Sleeve

EN	ELBOW ORTHOSIS
DE	ELLENBOGENORTHESE
FR	ORTHÈSE DU COUDE
ES	ORTESIS DE CODO
IT	ORTESI PER GOMITO
NL	ELLEBOOGORTHESE
DK	ALBUEORTOSE
FI	KYYNÄRNIVELORTOOSI
SE	ARMBÅGSORTOS
NO	ORTOSE, ALBUE
PL	ORTEZA ŁOKCIA
GR	ΝΑΡΘΗΚΑΣ ΑΓΚΩΝΑ

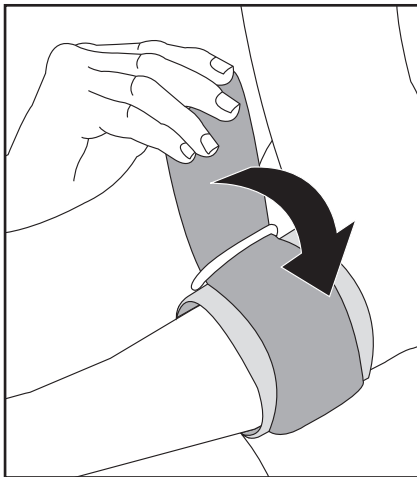

### ENGLISH

#### Product Description

Elbow strap which applies pressure to the forearm and elbow to ease the symptoms of tennis elbow and more.

#### Usage Information (See diagram)

1. Position strap just below elbow and loop strap around forearm.
2. Secure strap by pulling strap through D-Ring.
3. Pull strap back on itself and secure using hook/loop fastening. Readjust for comfort, ensuring a firm fit without being tight.

#### Indications for Use

- Tennis Elbow.
- Golfers Elbow.
- Tendonitis.
- Epicondylitis.
- Bursitis.

#### Contraindications

- Any injury requiring orthopaedic treatment with a plaster cast, resin cast or surgery.
- The strap must not be in direct contact with a skin lesion.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

#### Product Family Composition

Nylon, SBR.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

**MD** MEDICAL DEVICE

**1** SINGLE PATIENT - MULTIPLE USE

**!** CAUTION



Promedics Orthopaedics Ltd.  
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,  
Scotland, UK, PA14 5XH  
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk  
www.promedics.co.uk

**CE** **EMERGO EUROPE**  
Westervoortsedijk 60,  
6827 AT Arnhem  
The Netherlands