
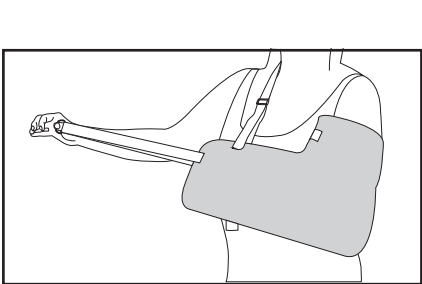
EVS - Elevator Sling

EN	ELEVATION ARM SLING
DE	ARMHEBESCHLINGE
FR	SANGLE D'ÉLEVATION DU BRAS
ES	ESLINGA PARA ELEVACIÓN DE BRAZO
IT	FASCIA PER SOLLEVAMENTO DEL BRACCIO
NL	SLING VOOR VERHOOGING VAN DE ARM
DK	ARMELEVATIONSSLYNGE
FI	KÄSIVARREN KOHOTUSSIDE
SE	ARMSLYNGA, UPPHÄNGNINGSBAR
NO	FATLE, ARM, HEVET
PL	TEMLAK DO PODNOSZENIA RAMIENIA
GR	ANAPTHΞH ANΩ AKPOY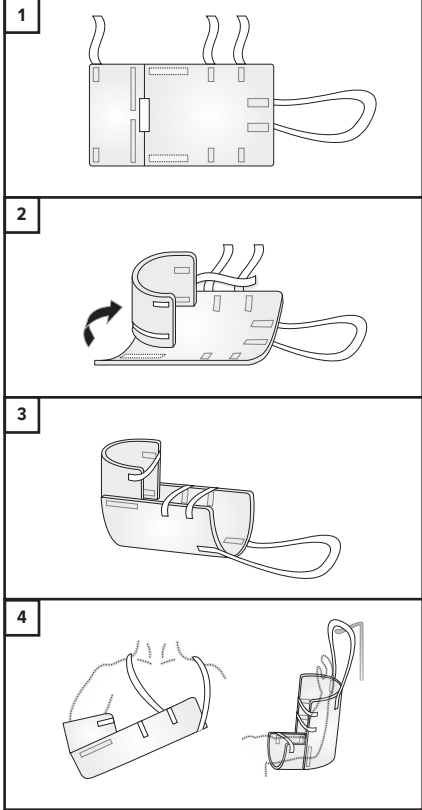

ENGLISH

Product Description

Universal sling that can be used as a traditional sling or arm elevation following trauma, burns or for post operative elevation of the arm.

Usage Information (See diagram)

- Lay flat.
- To assemble, fold then attach strips.
- Zusammengebaut.
- Use as traditional sling with strap around neck, or to elevate the arm.
- Angle can be adjusted using the hook/loop tabs.
- Paediatric version shown. Note, the angle of the paediatric version cannot be adjusted.

Indications for Use

- Breaks.
- Strains / sprains.
- Burns.
- Repetitive Injuries.
- Post Surgery.

Contraindications

- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash sling in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Cotton, Nylon, Polyester, Polyurethane, PU Foam, Tafetta, Steel.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.
Promedics Orthopaedics Ltd.