FA - Fixed Ankle

EN ANKLE/FOOT ORTHOSIS

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ES ORTESIS DE TOBILLO/PIE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORTHESE

DK ANKEL-/FODORTOSE

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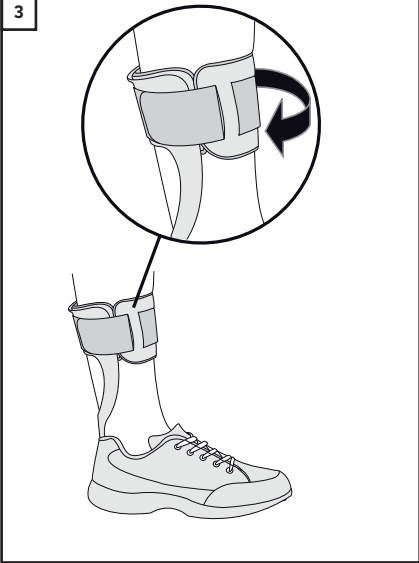
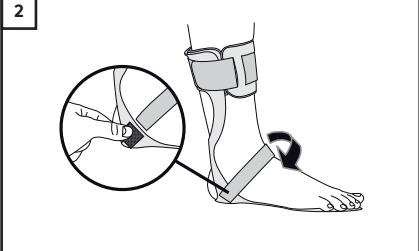
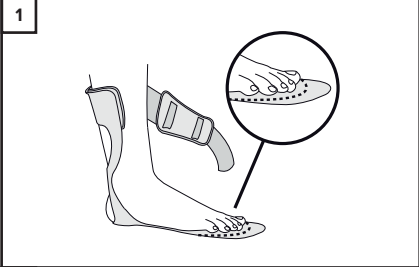
SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

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ENGLISH

Product Description

A lightweight, drop foot splint which provides dorsiflexion assistance and prevents plantarflexion.

Usage Information (See diagram)

- Place foot into the brace, ensuring heel is located to back. Foot section length can be easily trimmed to fit shoe. REMOVE brace from patient and trim with scissors ensuring all edges are smooth.
- Fasten hook / loop strap for a snug but comfortable fit. Do not over tighten.
- Ideally fit brace over item of clothing.

Caution:

Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions. If any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.

Indications for Use

- Weakness of dorsiflexion of foot (Drop foot).

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash shoe in cool water using mild detergent.
- Rinse and air dry.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Carbon, Nylon, Polyester, Polypropylene, PU Foam.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.