## HC - Heel Cup

**EN** ORTHOTIC INSOLE, PREFABRICATED, NON-ANTIMICROBIAL  
**DE** SCHUHEINLEGEORTHESE.

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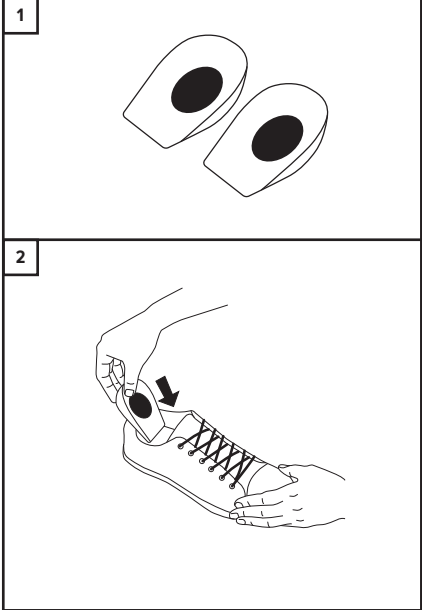
**ENGLISH**

**Product Description**

Pressure sensitive heel support for comfort.

**Usage Information**

1. Place into the shoe, ensuring heel is located to back of shoe.

**Indications for Use**

• Reducing pressure, friction and shear force on the heels.

**Contraindications**

• Any known allergy and/or hypersensitivity to any material listed in product composition.

**Warnings and Precautions**

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

**Washing and Care Instructions**

Hand wash brace in cool water using mild detergent.

**Storage and Transport Conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

**Recycling and Disposal**

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

**Serious Incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2024 Promedics Orthopaedics Ltd. All Rights Reserved.