HUP - Heal-up Protection Boot

| | |
|----|-------------------------------|
| EN | HEEL PROTECTOR |
| DE | FERSENSCHUTZ |
| FR | PROTÈGE-TALON |
| ES | PROTECTOR PARA TALÓN |
| IT | PROTEZIONE PER TALLONE |
| NL | HIELBESCHERMER |
| DK | HÆLBESKYTTELSESUDSTYR |
| FI | KANTAPÄÄSUOJUS |
| SE | HÅLSKYDD |
| NO | BESKYTTER, HÆL |
| PL | OCHRANIACZ PIĘTY |
| GR | ΠΡΟΣΤΑΤΕΥΤΙΚΟ ΠΤΕΡΝΑΣ |

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT


HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

HEAL-UP PROTECTION BOOT

Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

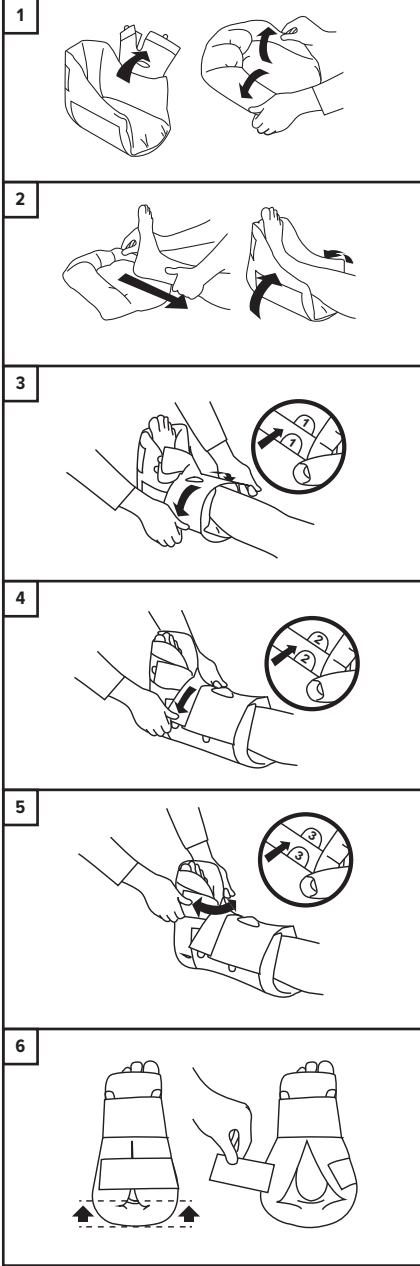ENGLISH

Product Description

A cushioned boot orthosis to provide effective heel off-loading, with open heel for increased inspection visibility.

Usage Information (See diagram)

- Open up the straps and fold the sides down flat / 'inside-out'.
- Slide flat under the heel, then fold the sides back up around the foot / ankle.
- Tighten strap 1, towards corresponding number tab. Evenly on both sides. Do not over tighten.
- Tighten strap 2, towards corresponding number tab. Evenly on both sides. Do not over tighten.
- Tighten strap 3, towards corresponding number tab. Evenly on both sides. Do not over tighten.
- Make sure the heel is floated. Open heel provides greater inspection visibility.

Indications for Use

- Heel pressure ulcers.
- Maintaining heel suspension.
- Preventing of foot drop and malpositioning.
- Reducing pressure, friction and shear force on the heels.
- Separating and protecting the ankles.

Contraindications

- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- The Heal –Up Heel Relief Boot can be machine washed at 40°C WOOL CYCLE using a mild detergent. DO NOT USE FABRIC CONDITIONER, DO NOT USE BLEACH.
- Ensure all hook and loop straps are closed and no hook is exposed. This avoids fabric particles damaging the hook closures.
- After removing from machine, open all straps and air-dry on a clothes horse or clothes airer. Keep away from any heat sources.
- When completely dry ensure that Heal- Up filling is evenly distributed throughout the boot. Do not iron, do not dry clean.
- Check hook closures and clear any build up of fabric or lint as this can affect the ability to fasten straps securely and the overall performance of the Heal-Up.

Product Family Composition

Polyester, Nylon, PVC.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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