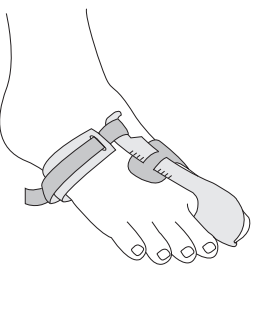
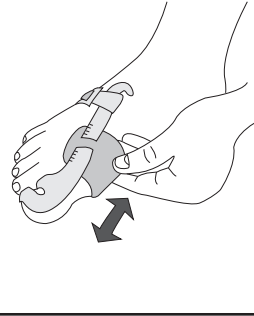
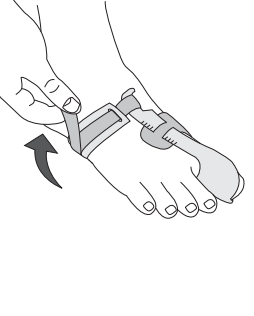
HVN - Hallux Valgus Night Splint

EN	FOOT ORTHOSIS
DE	FUßORTHESE
FR	ORTHÈSE DU PIED
ES	ORTESIS DE PIE
IT	ORTESI PER PIEDE
NL	VOETORTHESE
DK	FODORTOSE
FI	JALKATERÄORTOOSI
SE	FOTORTOS
NO	ORTOSE, FOT, ORDINÆR
PL	ORTEZA STOPY
GR	ΟΡΘΩΣΗ ΠΟΔΙΟΥ

1	
2	
3	

ENGLISH

Product Description

A universal size splint that helps maintain the large toe in a neutral position.

Usage Information (See diagram)

Initially apply the Hallux Valgus Night Splint using light pressure on the big toe, then gradually increase over time.

Open the hook/loop closure. Slip big toe into the splint. The integrated pad should surround the big toe and be flush with the tip of the toe.

The splint should be positioned approx. along the midline of the foot. The closer the pad is placed to the big toe along the serrated edge of the splint, the stronger the pressure applied to the big toe.

Fasten the strap, adjusting pressure to the big toe.

Please note: Splint may not be used in shoe.

Indications for Use

• Hallux Valgus (Bunion).

Contraindications

• Do not use over open wounds.
• Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over hook/loop strap .
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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