
## IWB - Intrinsic Wrist Brace

EN	<b>WRIST/HAND ORTHOSIS</b>
DE	<b>HANDELENK-/HANDORTHESE</b>
FR	<b>ORTHÈSE DU POIGNET/DE LA MAIN</b>
ES	<b>ORTESIS DE MUÑECA/MANO</b>
IT	<b>ORTESI PER POLSO/MANO</b>
NL	<b>POLS-HANDORTHESE</b>
DK	<b>HÄNDLEDS-/HÄNDORTOSE</b>
FI	<b>RANNE-KÄMMENORTOOSI</b>
SE	<b>HÄNDLEDS- OCH HANDORTOS</b>
NO	<b>ORTOSE, HÄNDLEDD/HÅND</b>
PL	<b>ORTEZA NADGARSTKA/DŁONI</b>
GR	<b>ΝΑΡΘΗΚΑΣ ΚΑΡΠΟΥ/ΧΕΙΡΟΥ</b>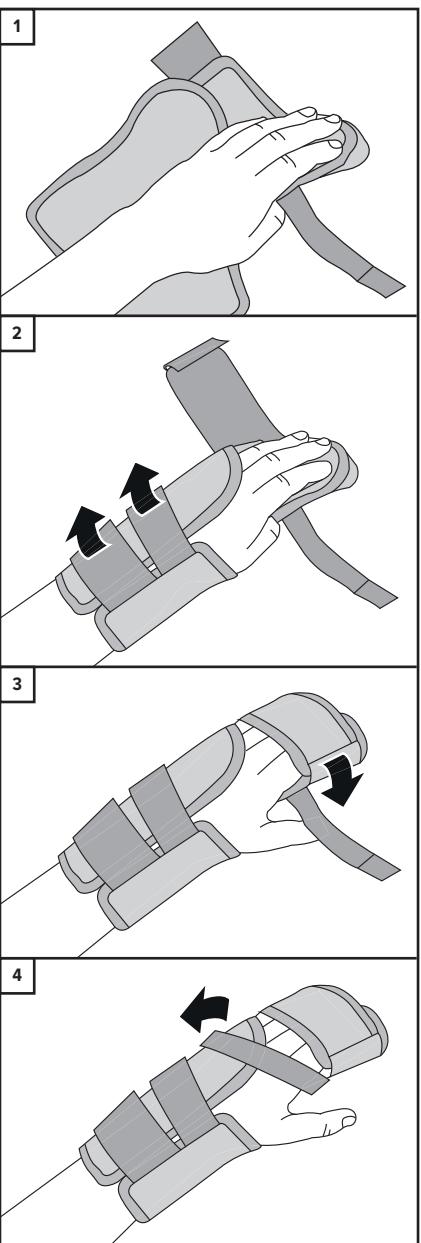

### ENGLISH

#### Product Description

Padded elastic wrist brace with a full-length adjustable aluminium resting splint. The splint is wide enough to contain all four fingers and can be easily adjusted to hold the wrist and fingers in the desired position. There is a wide, padded adjustable dorsal closure strap that can be used to hold the fingers in position.

#### Usage Information (See diagram)

- Open hook / loop fastening and open brace out flat. Position hand in brace as shown. Adjust the malleable aluminium splint to rest 4 fingers at the chosen functional angle.
- Close hook / loop fastening.
- Close hook / loop fastening securely around fingers.
- Close hook / loop fastening. Re-adjust fastenings for comfort as required ensuring metal stay is in position.

#### Indications for Use

- Osteoarthritis.
- Post carpal/metacarpal surgery.
- Supports CMC/MCP joints.
- Flaccid paralysis.
- Flexor tendon injury.

#### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

- Close over hook/loop straps.
- Remove metal splint.
- Hand wash orthosis in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fastener is operating securely.
- Replace metal splint.

#### Product Family Composition

Aluminium, Elastane, Nylon, Polyamide, Polyester.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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