
JPC - Palm Cover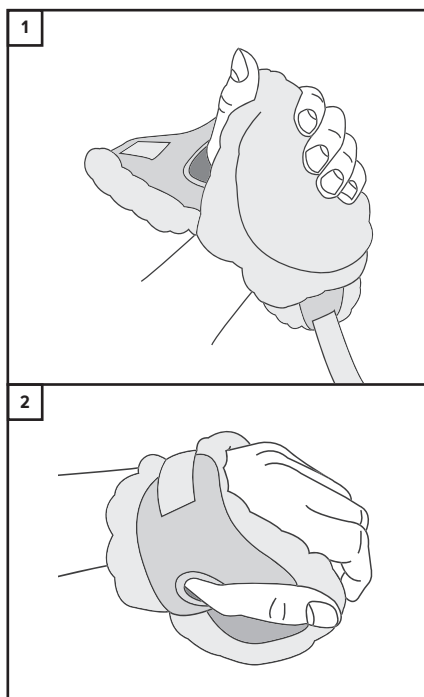

EN	HAND ORTHOSIS
DE	HANDORTHESE
FR	ORTHÈSE DE LA MAIN
ES	ORTESIS DE MANO
IT	ORTESI PER MANO
NL	HANDORTHESE
DK	HÅNDORTOSE
FI	KÄMMENORTOOSI
SE	HANDORTOS
NO	ORTOSE, HÅND
PL	ORTEZA DŁONI
GR	ΝΑΡΘΗΚΑΣ ΧΕΙΡΟΣ

ENGLISH

Product Description

Designed to allow full finger movement whilst reducing finger contractures and skin damage/breakdown in the palmar region.

Usage Information (See diagram)

1. Unhook the hook and loop strap and fit the Jura Palm Cover with patients thumb through thumb hole as shown. (Fig. 1) The Jura label should be visible on the outside.
2. Pull the loop strap across the back of patient's hand and fit securely in place with hook tab. Trim strap length if necessary. (Fig. 2)
3. To extend the fingers further and help release a tight grip, insert cylindrical foam or a cloth between the palm and finger parts of the Jura Palm Cover.
4. Note: Ensure hook tab is not in contact with skin. If there is contact, the Jura Palm Cover has been fitted inside-out.

Indications for Use

Palm Protectors can offer relief from curling fingers, hand contractures and cramping. The Palm Protector provides a cushioning pad for the fingers to curl onto, preventing the finger tips/nails from digging into the palms, whilst also keeping the fingers supported.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
 - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over hook/loop strap.
2. Hand wash in cool water using mild detergent.
3. Rinse and air dry.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fastener is operating securely.

Product Family Composition

Nylon, Polyester.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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1 SINGLE PATIENT - MULTIPLE USE

! CAUTION

Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44(0)1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **EC** **REP** **EMERGO EUROPE**
Westervoortsewijk 60,
6827 AT Arnhem
The Netherlands

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