
KI - Knee Immobiliser

EN	KNEE ORTHOSIS
DE	KNIEORTHESE
FR	ORTHÈSE DU GENOU
ES	ORTESIS DE RODILLA
IT	ORTESI PER GINOCCHIO
NL	KNIEORTHESE
DK	KNÆORTOSE
FI	POLVIORTOSI
SE	KNÅORTOS
NO	ORTOSE, KNE, ORDINÆR
PL	ORTEZA STAWU KOLANOWEGO
GR	ΝΑΡΘΗΚΑΣ ΓΟΝΑΤΟΣ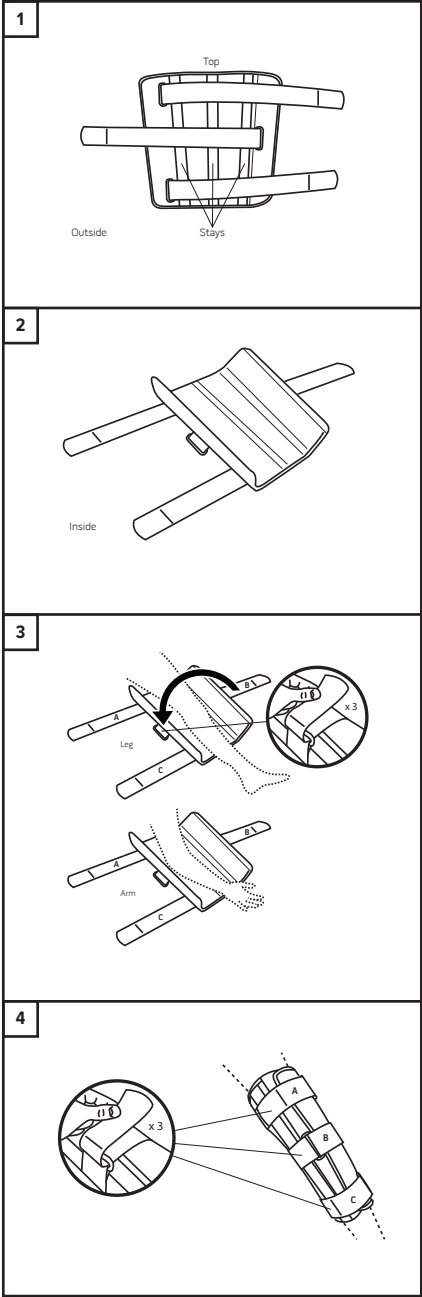

ENGLISH

Product Description

An immobilisation orthosis with straps and stays to help immobilise the joint.

Usage Information (See diagram)

- Remove the outer packaging and open all the straps on the immobiliser.
- Lay the immobiliser flat with the buckles and straps facing downwards. The immobilisation stays can be reshaped if required.
- Slide the brace under patient's limb.
- Fasten straps to secure the brace to the patient.

Indications for Use

- Conservative treatment for knee / arm injuries.

Contraindications

- Do not apply in direct contact with broken skin.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
 - Follow all instructions to ensure proper performance of the device.
 - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
 - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
 - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
 - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
 - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Aluminium, Cotton, Elastane, Nylon, Polyamide, Polyester, POM, PU Foam, SBR, Spandex, Steel.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.