

**LAB - Lincoln Ankle Brace**

EN ANKLE/FOOT ORTHOSIS

DE FUßGELENK-/FUßORTHSE

FR ORTHÈSE DE CHEVILLE/PIED

ES ORTESIS DE TOBILLO/PIE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORHSE

DK ANKEL-/FODORTOSE


FI NILKKA-JALKATERÄORTOOSI

SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

GR ΝΑΡΘΗΚΑΣ ΑΣΤΡΑΓΑΛΟΥ/ΠΟΔΙΟΥ

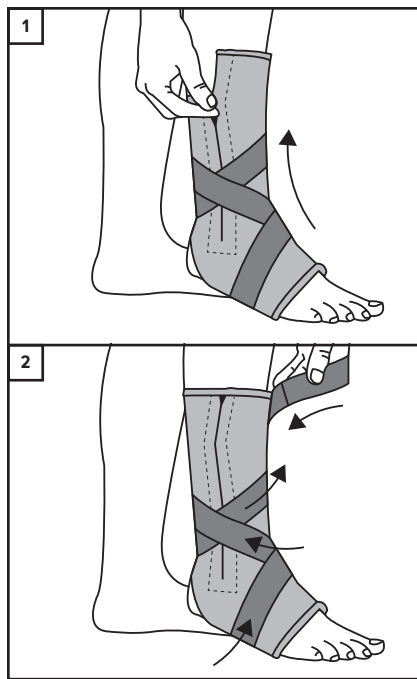

**ENGLISH**

**Product Description**

A zipped ankle support in Ventoprene, which is ventilated, towelling – lined Neoprene.

**Usage Information (See diagram)**

1. Place the foot into the closed section of the ankle support. Pull the zip up to close the ankle support.
2. Fasten the loose straps around the ankle support in a figure of 8 motion as shown in the diagram.

**Indications for Use**

- Ankle instabilities.

**Contraindications**

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

**Warnings and Precautions**

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

**Washing and Care Instructions**

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

**Product Family Composition**

Nylon, SBR, Steel, Polyamide, Elastane, Polyester Acetal, Zinc, Stainless Steel, Aluminium.

**Storage and Transport Conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

**Recycling and Disposal**

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

**Serious Incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

**MD** MEDICAL DEVICE

**1** SINGLE PATIENT - MULTIPLE USE

**!** CAUTION



Promedics Orthopaedics Ltd.  
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,  
Scotland, UK, PA14 5XH  
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk  
www.promedics.co.uk

**CE** **EC** **REP** **EMERGO EUROPE**  
Westervoortsewijk 60,  
6827 AT Arnhem  
The Netherlands