


NSB - Neoprene Shoulder Brace

EN	SHOULDER ORTHOSIS
DE	SCHULTERORTHESE
FR	ORTHÈSE DE L'ÉPAULE
ES	ORTESIS DE HOMBRO
IT	ORTESI PER SPALLA
NL	SCHOUDERORTHESE
DK	SKULDERORTOSE
FI	OLKANIVELORTOOSI
SE	AXELORTOS
NO	ORTOSE, SKULDER
PL	ORTEZA STAWU BARKOWEGO
GR	ΝΑΡΘΗΚΑΣ ΩΜΟΥ

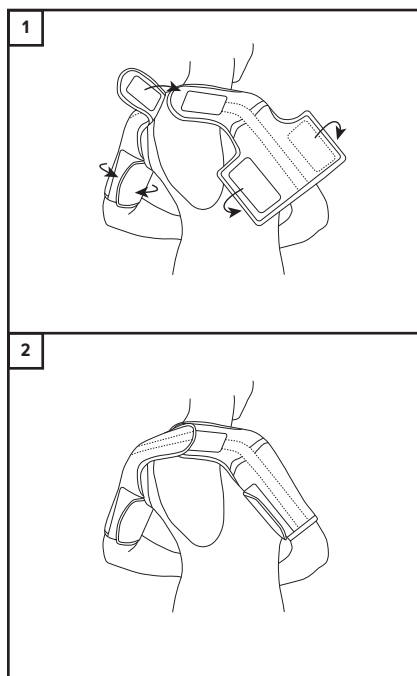

ENGLISH

Product Description

Two adjustable upper arm cuffs fix with hook/loop at the back of the neck, to prevent shoulder subluxation and hold the head of humerus in the glenoid cavity.

Usage Information (See diagram)

1. Place upper arm into cuff.
2. Secure the hook & loop fastening.
3. Take across back of neck.
4. Place opposite arm into second cuff and secure the hook & loop fastening.
5. Fasten the hook and loop behind the neck.
6. To achieve more abduction undo the hook and loop at rear of the neck, tighten and re-secure.

Indications for Use

- Shoulder instability.
- Lax capsule.
- Rhomboid strain.
- Trapezius strains.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Nylon, SBR, Polyamide, Elastane.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

1 SINGLE PATIENT - MULTIPLE USE

! CAUTION



Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **EC** **REP** **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands