
## OHS - Off-loader Heel Shoe

EN	<b>FOOT ORTHOSIS</b>
DE	<b>FUßORTHESE</b>
FR	<b>ORTHÈSE DU PIED</b>
ES	<b>ORTESIS DE PIE</b>
IT	<b>ORTESI PER PIEDE</b>
NL	<b>VOETORTHESE</b>
DK	<b>FODORTOSE</b>
FI	<b>JALKATERÄORTOOSI</b>
SE	<b>FOTORTOS</b>
NO	<b>ORTOSE, FOT, ORDINÆR</b>
PL	<b>ORTEZA STOPY</b>
GR	<b>ΟΡΘΩΣΗ ΠΟΔΙΟΥ</b>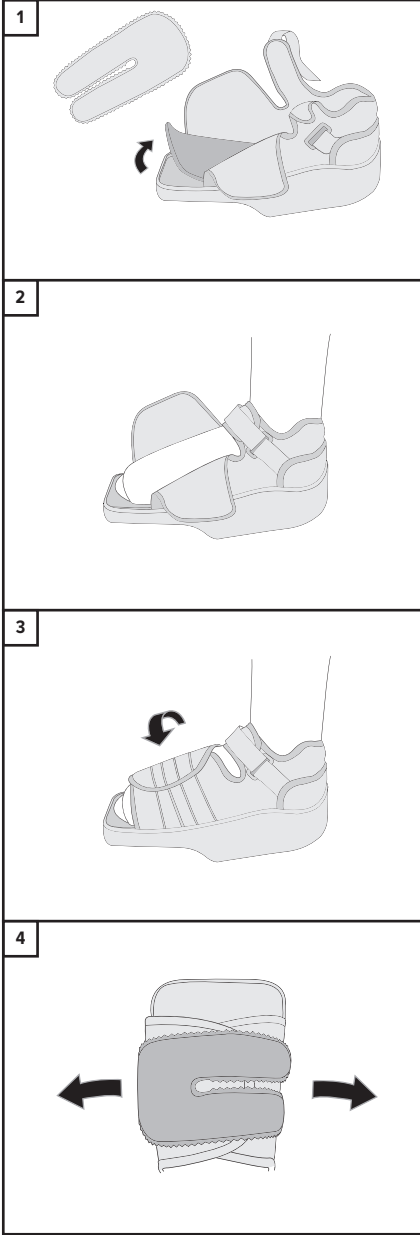

### ENGLISH

#### Product Description

Square toe off loading shoe designed for relief of the forefoot.

#### Usage Information (See diagram)

- 1) Open up shoe.
- 2) Position foot, ensuring heel is located to back of shoe and fasten ankle strap.
- 3) Close over instep wraps and fix securely in place using the adjustable closure. This can be adjusted to accommodate dressings etc.
- 4) Insole can be removed and replaced with off-loading custom insole.

#### Indications for Use

- Post-Surgical healing.
- Forefoot trauma.
- Wounds or ulcerations present under metatarsal heads and toes.
- Treatment of Forefoot ulcerations.
- Any situation that requires pressure to be off-loaded of the forefoot.
- Used post-operatively after procedures to correct hallux valgus, hammer toes, tailor’s bunions.
- Treatment of Forefoot ulcerations.

#### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

#### Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
  - Follow all instructions to ensure proper performance of the device.
  - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
  - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
  - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
  - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
  - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

- 1) Hand wash shoe with cloth using water and mild detergent.
- 2) Allow the shoe to air dry.
- 3) Do not spin or tumble dry.
- 4) Do not place on radiator or heater.
- 5) Do not use fabric conditioner at any time.

#### Product Family Composition

Polyester, Polyurethane, Nylon, POM, EVA, Cellulose, TPR.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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Promedics Orthopaedics Ltd.