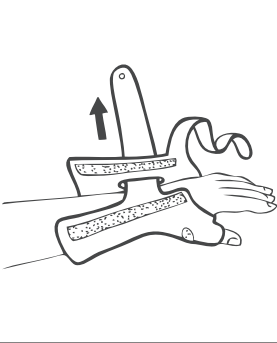
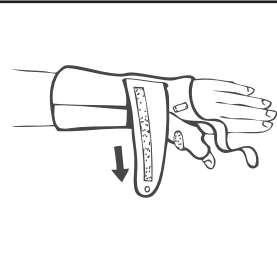
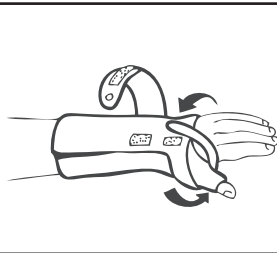
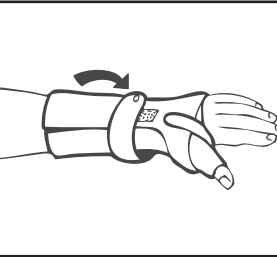
## PWTRS - Procool Wrist Thumb Restriction Brace

EN	<b>WRIST/HAND ORTHOSIS</b>
DE	<b>HANDGELENK-/HANDORTHESE</b>
FR	<b>ORTHÈSE DU POIGNET/DE LA MAIN</b>
ES	<b>ORTESIS DE MUÑECA/MANO</b>
IT	<b>ORTESI PER POLSO/MANO</b>
NL	<b>POLS-HANDORTHESE</b>
DK	<b>HÅNDELED-/HÅNDORTOSE</b>
FI	<b>RANNE-KÄMMENORTOOSI</b>
SE	<b>HANDLEDS- OCH HANDORTOS</b>
NO	<b>ORTOSE, HÅNLEDD/HÅND</b>
PL	<b>ORTEZA NADGARSTKA/DŁONI</b>
GR	<b>ΝΑΡΘΗΚΑΣ ΚΑΡΠΟΥ ΎΙΧΕΡΙΟΎ</b>

<b>1</b>	
<b>2</b>	
<b>3</b>	
<b>4</b>	

#### ENGLISH

##### Product Description

A wrist/thumb orthosis which provides adjustable support to the thumb carpometacarpal (CMC) and metacarpal phalangeal (MCP) joint.

##### Usage Information (See diagram)

- Position hand in brace.
- Close hook/loop fastening around the thumb.
- Close the hook/loop fastening around the wrist.
- Do not tighten too much, re-adjust for comfort as required while ensuring splint is in correct position.

##### Indications for Use

- Provides thumb CMC and MCP joint support for an array of injuries, conditions, pre- and post-operations such as:
  - Arthritis.
  - Sprains and strains.
  - Soft tissue injuries and conditions.
  - Instability.

##### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

##### Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
  - Follow all instructions to ensure proper performance of the device.
  - Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
  - Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
  - Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
  - Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
  - The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Close over all Hook/Loop straps.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check Hook/Loop fasteners are operating securely.

##### Product Composition

Nylon, SBR, Polyester.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.