ROMW - ROM Walker

EN ANKLE/FOOT ORTHOSIS

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ES ORTESIS DE TOBILLO/PIE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORTHESE

DK ANKEL-/FODORTOSE

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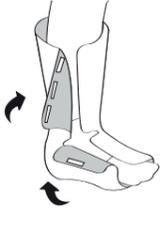
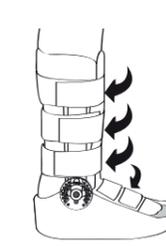
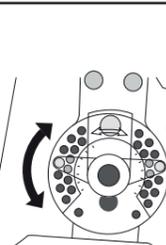
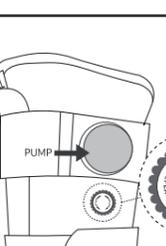
SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

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ENGLISH

Product Description

Lightweight walker boot for immobilisation with range of movement hinge for plantarflexion or dorsiflexion.

Usage Information (See diagram)

- When applying walker for first time open all straps and remove liner. Apply liner ensuring the heel is pushed fully into the liner. Fasten hook closures on foot and leg sections ensuring they are snug.
- Pull uprights apart and place foot/liner into walker ensuring heel is against posterior border of walker. Uprights should be aligned with mid ankle and lower leg. Apply pressure to ensure uprights are fixed to liner.
- Fasten straps, from toes up, ensuring they are firm but not too tight.
- The ROM Joint is adjustable in 7.5 degree increments for both dorsi and plantar flexion up to a maximum of 45 degrees. To obtain required setting, pull out pins and locate in the pin hole that corresponds with desired position. The settings are clearly marked. The joint can be locked at the desired setting or a selected range setting can be used. If the joint is being locked in a desired setting, the hex screw must be tightened down with the provided hex key.
- Air liner version – To INFLATE air bags, turn dial to “INFLATE” and press the pump. To DEFLATE airbags, turn dial to “DEFLATE” and push air from air bags.

Indications for Use

- Post-Surgery rehabilitation.
- Stable fracture of fibula, talus, calcaneus and malleolus.
- Controlled rehabilitation following Achilles tendon surgery.

Contraindications

- Not intended for unstable fractures.
- Do not apply directly to broken skin.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Remove liner from boot.
- Hand wash liner in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Wipe down the shell with clean damp cloth, mild detergent and allow to air dry.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Aluminium, EVA, Glass Fibre, Nickel, Nylon, Polyamide, Polycarbonate, Polyester, Polyethylene, Polyolefin, Polypropylene, Polyurethane, POM, PU Foam, Steel, TPR, TPU.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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