


SPD - Abdominal Binder

EN ABDOMINAL BINDER, REUSABLE
DE ABDOMINALBINDE, WIEDERVERWENDBAR
FR BANDAGE ABDOMINAL, RÉUTILISABLE
ES FAJA ABDOMINAL, REUTILIZABLE
IT FASCIA ADDOMINALE, RIUTILIZZABILE
NL BUIKBANDAGE, HERBRUIKBAAR
DK STØTTEBIND TIL ABDOMEN, GENANVENDELIGT
FI VATSAN TUKISIDE, UUDELLEEN KÄYTETTÄVÄ
SE BUKBINDA, FLERGÅNGS
NO BIND, STØTTE, ABDOMINALT, FLERGANGSBRUK
PL BRZUSZNY PAS PODTRZYMUJĄCY, WIELOKROTNEGO UŻYTKU
GR ΖΩΝΗ ΚΟΙΝΙΑΣ, ΕΠΑΝΑΧΡΗΣΙΜΟΠΟΙΟΥΜΕΝΗ

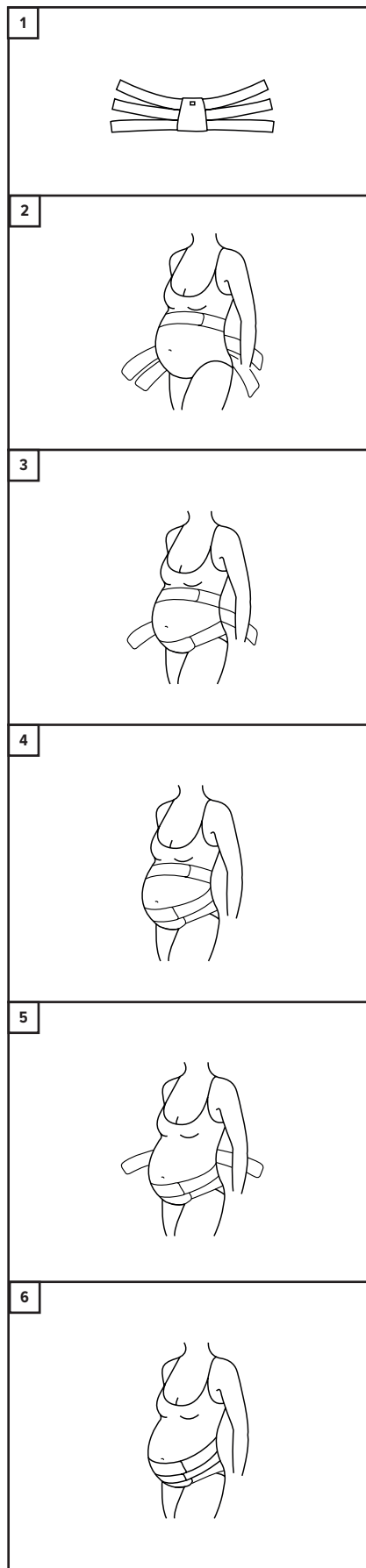

ENGLISH

Product Description

Pelvic instability belt with 'many tailed' pattern allows variable tension on each set of straps, with strong tension over pubic area and lighter tension under the abdomen. Washing instructions include sling.

Usage Information (See diagram)

1. Open the belt out and place it on the lower back. The wide portion of the belt (label inside and at the top) should fit snugly into the hollow of the back.
2. Anchor the belt temporarily by drawing the top tails around the upper abdomen and fastening together loosely.
3. Bring the lowest tails downwards and forwards under the lower abdomen across the pubic bone, tightening as much as necessary to give support without pressing unduly on the symphysis pubis. Fasten together.
4. Bring the middle tails around in the same way fastening together just above and slightly overlapping the lowest pair.
5. Adjust the top tails again in a downward and forwards direction to slightly overlap the middle pair. Do not wear the top tails at breastbone level.
6. This illustrates the final position of all tails overlapping to give full support to the pelvis.

Indications for Use

The brace is designed for expectant mothers. Allows variable tension on each set of straps, with strong tension over the pubic area and lighter tension under the abdomen. Adaptable to a standing or sitting position and can be worn over clothing. Provides strong circumferential support around the pelvis holding the two sacroiliac joints and the symphysis pubis joint firmly. It is recommended that individuals seek guidance from a medical practitioner regarding the most appropriate brace and wearing schedule.

Contraindications

- Unsuitable for prolonged use.
- Care is required to ensure that the garment is not constrictive.
- Any known allergy and/ or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Composition

Nylon, Polyester, Elastane, Cotton.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

1P SINGLE PATIENT - MULTIPLE USE

! CAUTION



Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **REP** **EMERGO EUROPE**
Westervoortsewijk 60,
6827 AT Arnhem
The Netherlands