


TB - Trochanteric Belt

EN	HIP ORTHOSIS
DE	HÜFTORTHESE
FR	ORTHÈSE DE LA HANCHE
ES	ORTESIS DE CADERA
IT	ORTESI PER ANCA
NL	HEUPORTHESE
DK	HOFTEORTOSE
FI	LONKKAORTOSI
SE	HÖFTORTOS
NO	ORTOSE, HOFTE
PL	ORTEZA STAWU BIODROWEGO
GR	ΝΑΡΘΗΚΑΣ ΙΣΧΙΟΥ

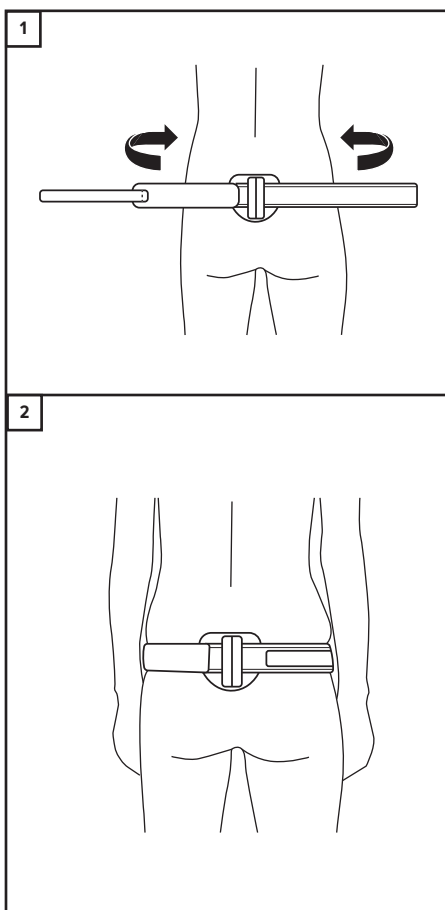

Product Description

A strong webbing strap with foam padding gives firm circumferential support at trochanteric level to ease the pain of separated symphysis. Can also be used to treat sport-induced symphysis pubis pain.

Usage Information (See diagram)

1. Wrap the Trochanteric Belt around the middle of the hips (midway between the crest of the pelvis and the hip socket).
2. Fasten the belt to be firm but not tight.

Indications for Use

- General pelvic instability during pregnancy.
- Sacroiliac joint pain.
- Symphylolysis.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the brace is not being worn or when being washed.

Washing and Care Instructions

1. Close over all hook/loop straps.
2. Hand wash in cool water using mild detergent.
3. Rinse and dry flat.
4. Do not spin or tumble dry.
5. Do not use fabric conditioner at any time.
6. Check for any degradation that may affect use.
7. Periodically check hook/loop fasteners are operating securely.

Product Composition

Nylon, Polyester, Steel.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

1 SINGLE PATIENT - MULTIPLE USE

! CAUTION



Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **EC** **REP** **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands