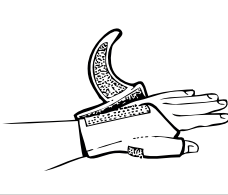
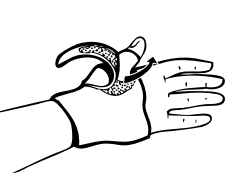
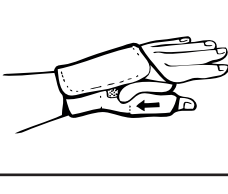
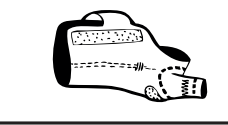
TRS - Thumb Restriction Splint

EN	HAND ORTHOSIS
DE	HANDORTHESE
FR	ORTHÈSE DE LA MAIN
ES	ORTESIS DE MANO
IT	ORTESI PER MANO
NL	HANDORTHESE
DK	HÅNDORTOSE
FI	KÄMMENORTOOSI
SE	HANDORTOS
NO	ORTOSE, HÅND
PL	ORTEZA DŁONI
GR	ΝΑΡΘΗΚΑΣ ΧΕΙΡΙΟΥ

1	
2	
3	
4	

ENGLISH

Product Description

Gentle compression to relieve thumb discomfort associated with CMC joint arthritis.

Usage Information (See diagram)

- Slide the thumb into the thumb sleeve. Wrap the splint around the wrist and fasten the hook/loop on top of the hand so that the brace feels comfortable.
- Align the strap over the dorsal CMC joint and fasten to the curved hook/loop piece. Pull the CMC strap along the curved hook/loop piece and around the CMC joint.
- Place the strap through the thumb web space and fasten to the hook/loop piece on top of the hand. Ensure the strap feels comfortable in the web space and trim strap if required.
- For greater thumb IP motion, cut along the volar thumb edge. If the thumb MP joint support is not required, cut of below the thumb web seam.

Indications for Use

- CMC joint arthritis.
- CMC joint instability.
- Tendonitis.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.
Promedics Orthopaedics Ltd.