TSL - Thumb Spica

EN HAND ORTHOSIS

DE HANDORTHESE

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IT ORTESI PER MANO

NL HANDORTHESE

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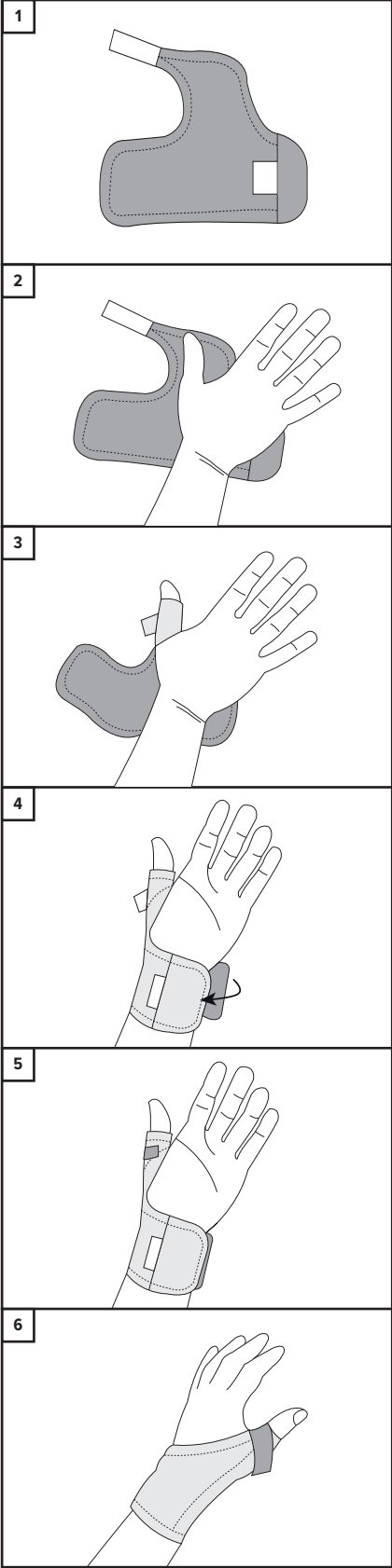
FI KÄMMENORTOOSI

SE HANDORTOS

NO ORTOSE, HÅND

PL ORTEZA DŁONI

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ENGLISH

Product Description

A thumb orthosis which provides adjustable support to the thumb carpometacarpal (CMC) and metacarpal phalangeal (MCP) joint. A flexible removable stay restricts movement at the CMC and MCP joints, leaving the interphalangeal (IP) joint free.

Usage Information (See diagram)

- Open hook/loop fastening and open brace out flat.
- Position hand in brace (see image 2).
- Close hook/loop fastening around the thumb.
- Close the hook/loop fastening around the wrist.
- Do not tighten too much, re-adjust for comfort as required while ensuring splint is in correct position.

Indications for Use

- Provides thumb CMC and MCP joint support for an array of injuries, conditions, pre- and post-operations such as:
 - Arthritis.
 - Sprains and strains.
 - Soft tissue injuries and conditions.
 - Instability.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps & remove metal splint.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.
- Before using brace again, replace metal splint ensuring splint is correct way round and splint is tucked under edge / ‘lip’ of pocket.

Product Family Composition

Cotton, Elastane, Lycra, Natural Rubber, Nylon, Polyamide, Polyester, PVC, Stainless Steel, Steel, Viscose.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.
Promedics Orthopaedics Ltd.