## TSTP - Thumb Guard

### EN HAND ORTHOSIS

### DE HANDORTHESE

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### IT ORTESI PER MANO

### NL HANDORTHESE

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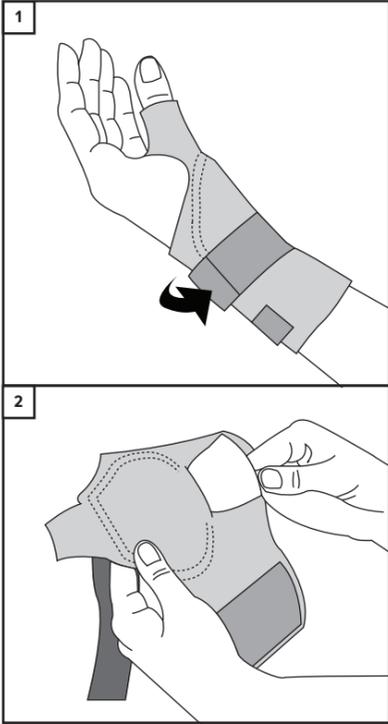
### FI KÄMMENORTOOSI

### SE HANDORTOS

### NO ORTOSE, HÅND

### PL ORTEZA DŁONI

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#### ENGLISH

##### Product Description

A thumb orthosis which provides adjustable support to the thumb carpometacarpal (CMC) and metacarpal phalangeal (MCP) joint.

##### Usage Information (See diagram)

##### Moulding directly to Thumb:

- Heat thermoplastic in water heated at approx 60° C, for 45 seconds, or until it becomes soft.
- Remove thermoplastic from water.
- Place on an absorbent surface to remove excess water. Check the temperature of the thermoplastic is safe for the patient.
- Mould the thermoplastic over the thumb.
- When the thermoplastic cools, insert into pocket of spica.
  - Return to first step if you need to re-adjust. Minor changes can be made with scissors.

##### Moulding within Brace:

- Insert thermoplastic into pocket of spica.
- Fold pocket over wide open, to ease pouring water into it.
- Pour 60° C water into pocket, filling it completely. If required, refill after 30 seconds. If thermoplastic is soft after approx 1 minute, pour out remaining water and place on an absorbent surface, to remove excess water from the spica.
- Check thermoplastic is at a safe temperature for the patient, apply spica and mould as required.
- Secure the thumb hook/loop strap and then wrist hook/loop straps.
- Leave spica on patient until thermoplastic cools and hardens, normally 3 - 5 minutes. Return to step 2 if you need to re-adjust. Minor changes can be made with scissors.

##### Indications for Use

• Provides thumb CMC and MCP joint support for an array of injuries, conditions, pre- and post-operations such as:

- Arthritis.
- Sprains and strains.
- Soft tissue injuries and conditions.
- Instability.

##### Contraindications

• Do not use over open wounds.  
• Any known allergy and/or hypersensitivity to any material listed in product composition.  
• Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

##### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

• Carefully read all instructions and warnings prior to use.  
• Follow all instructions to ensure proper performance of the device.  
• Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.  
• Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.  
• Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.  
• Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.  
• The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Close over all hook/loop straps & remove splint.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.
- Before using brace again, replace splint ensuring splint is correct way round and splint is tucked under edge / 'lip' of pocket.

##### Product Family Composition

Elastane, Nylon, Polyamide, Polyester, SBR, Spandex, Thermoplastic, Urethane.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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