
USHV - Universal Soft Hallux Valgus

EN	FOOT ORTHOSIS
DE	FUßORTHESE
FR	ORTHÈSE DU PIED
ES	ORTESIS DE PIE
IT	ORTESI PER PIEDE
NL	VOETORTHESE
DK	FODORTOSE
FI	JALKATERÄORTTOOSI
SE	FOTORTOS
NO	ORTOSE, FOT, ORDINÆR
PL	ORTEZA STOPY
GR	ΟΡΘΩΣΗ ΠΟΔΙΟΥ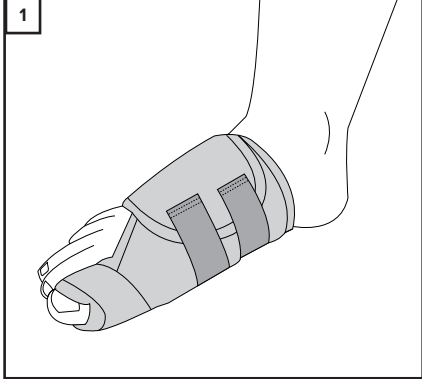

ENGLISH

Product Description

A soft padded universal size splint, with adjustable hook/loop fastening.

Usage Information (See diagram)
Position brace on foot as shown.

Indications for Use
• Assists in the positioning of the alignment of the large toe.

Contraindications
• Do not apply in direct contact with broken skin.
• Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
• Carefully read all instructions and warnings prior to use.
• Follow all instructions to ensure proper performance of the device.
• Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
• Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
• Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
• Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
• The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

<p>Product Family Composition Aluminium, Elastane, Nylon, Polyamide, Polyester, Polyurethane.</p>
--

Storage and Transport Conditions
Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal
Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident
Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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