


WFS - Whitechapel Finger Splint

EN	HAND/FINGER SPLINT, REUSABLE
DE	HAND-/FINGERSCHIENE, WIEDERVERWENDBAR
FR	GOUTTIÈRE POUR LA MAIN/LES DOIGTS, RÉUTILISABLE
ES	FÉRULA PARA DEDO/MANO, REUTILIZABLE
IT	SPLINT PER MANO/DITO, RIUTILIZZABILE
NL	HAND-/VINGERSPALK, HERBRUIKBAAR
DK	HÅND-/FINGERSKINNE, GENANVENDELIG
FI	KÄSI-/SORMILASTA, UUDELLEEN KÄYTETTÄVÄ
SE	HAND- OCH FINGERSKENA, FLERGÅNGS
NO	SKINNE, HÅND/FINGER, FLERGANGSBRUK
PL	SZYNA NA DŁON/PALEC, WIELOKROTNEGO UŻYTKU
GR	ΝΑΡΘΗΚΑΣ ΧΕΡΙΟΥ/ΔΑΚΤΥΛΟΥ, ΕΠΑΝΑΧΡΗΣΙΜΟΠΟΙΟΥΜΕΝΟΣ

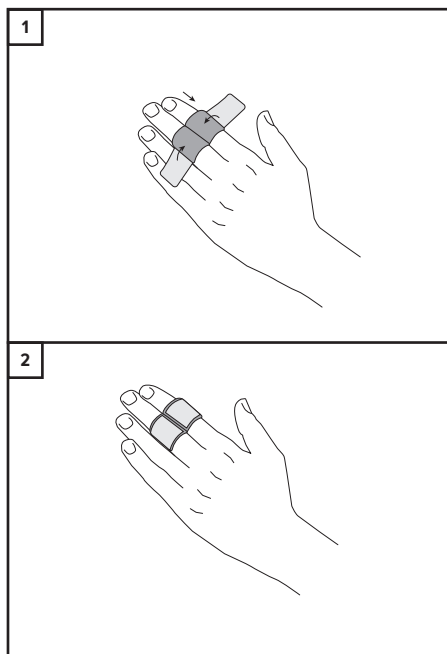

Product Description

To increase range of movement by securing the injured finger next to a stronger finger ('buddy strap' effect). Use for soft tissue injury to PIP joint.

Usage Information (See diagram)

- Slide fingers into splint and place over the PIP joints.
- Fold the straps back over themselves as shown in diagram 1.

Indications for Use

- Soft tissue injuries to PIP joint.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Polyamide, Nylon.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

1 SINGLE PATIENT - MULTIPLE USE

! CAUTION



Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

CE **REP** **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands