


WMB - Abdominal Binder

EN	ABDOMINAL BINDER, REUSABLE
DE	ABDOMINALBINDE, WIEDERVERWENDBAR
FR	BANDAGE ABDOMINAL, RÉUTILISABLE
ES	FAJA ABDOMINAL, REUTILIZABLE
IT	FASCIA ADDOMINALE, RIUTILIZZABILE
NL	BUIKBANDAGE, HERBRUIKBAAR
DK	STØTTEBIND TIL ABDOMEN, GENANVENDELIGT
FI	VATSAN TUKISIDE, UUDELLEEN KÄYTETTÄVÄ
SE	BUKBINDA, FLERGÅNGS
NO	BIND, STØTTE, ABDOMINAL T, FLERGANGSBRUK
PL	BRZUSZNY PAS POD TRZYMUJĄCY, WIELOKROTNEGO UŻYTKU
GR	ΖΩΝΗ ΚΟΙΛΙΑΣ, ΕΠΑΝΑΧΡΗΣΙΜΟΠΟΙΟΥΜΕΝΗ

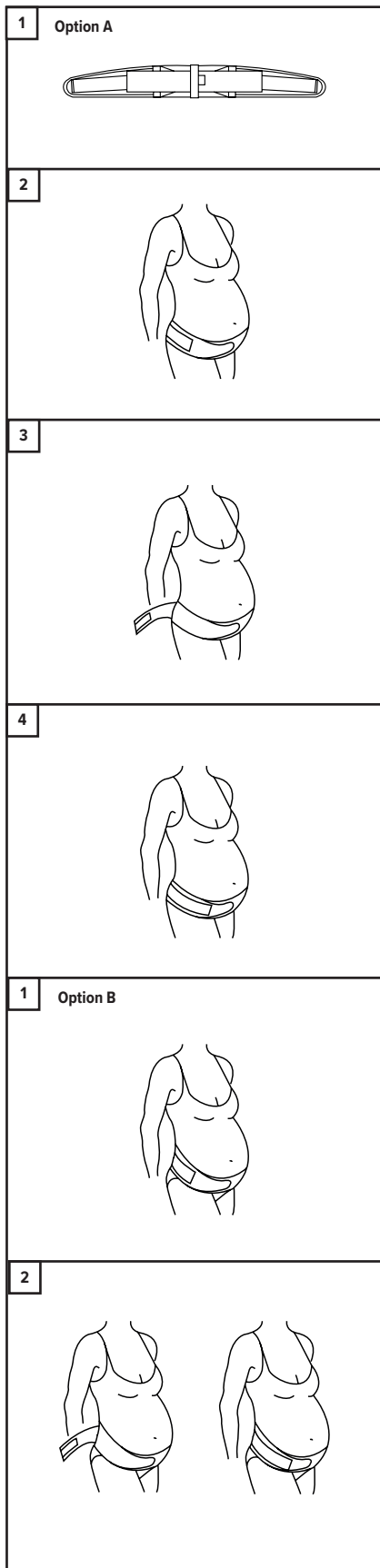

ENGLISH

Product Description

Designed to support the pelvic joints (the symphysis pubis joint at the front and the sacroiliac joints at the back), reducing strain on the joints and assisting pain relief. It can be used when standing and walking. It is not recommended for sitting or lying positions.

Usage Information (See diagram)

Option A:

- How to Apply - The belt can be used in either of the following positions (Option A - more support at the symphysis pubis or Option B - more support at the sacroiliac joints) according to the advice of your physiotherapist.
- Wrap the belt around the hip joints and close it using the Velcro fastening at the front.
- Grip the wide elasticated straps on the outside of the belt and draw them forward to tighten the belt so that it feels comfortable and supportive.
- The belt should lie below the abdomen, across the pubic bone at the front, and across the buttocks at the back.

Option B:

- Place the belt over the sacroiliac joints at the back and wrap the straps around the pelvis, directing them forwards and downwards to lie under the lower abdomen and across the pubic bone. Close the belt using the Velcro fastening at the front.
- Grip the wide elasticated straps on the outside of the belt and draw them forward to tighten the belt so that it feels comfortable and supportive.

Indications for Use

The device is designed for expectant mothers. With a slimline design the device is comfortable to wear sitting or standing and can be worn over or under clothes. The device provides:

- Strong support around the pelvis.
- Supports the two sacroiliac joints.
- Firm support of the symphysis pubis joint.
- Light tension under the abdomen.

Contraindications

- Unsuitable for prolonged use.
- Care is required to ensure that the garment is not constrictive.
- Any known allergy and/ or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash belt in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

MD MEDICAL DEVICE

 SINGLE PATIENT - MULTIPLE USE

 CAUTION



 Promedics Orthopaedics Ltd.
Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow,
Scotland, UK, PA14 5XH
Tel: +44 (0) 1475 746 400 | Email: enquiries@promedics.co.uk
www.promedics.co.uk

   **EMERGO EUROPE**
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

©2023 Promedics Orthopaedics Ltd. All Rights Reserved.

WMBIFUE revised 8/23