WTB - Wrist Thumb Brace

WTB - Wrist Thumb Brace

EN	WRIST/HAND ORTHOSIS
DE	HANDGELENK-/HANDORTHESE
FR	ORTHÈSE DU POIGNET/DE LA MAIN
ES	ORTESIS DE MUÑECA/MANO
IT	ORTESI PER POLSO/MANO
NL	POLS-HANDORTHESE
DK	HÅNDELED-/HÅDORTOSE
FI	RANNE-KÄMMENORTOOSI
SE	HANDLEDS- OCH HANDORTOS
NO	ORTOSE, HÅNLEDD/HÅND
PL	ORTEZA NADGARSTKA/DŁONI
GR	ΝΑΡΘΗΚΑΣ ΚΑΡΠΙΟΥ/ΧΕΙΡΙΟΥ

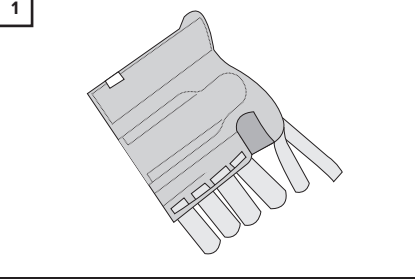
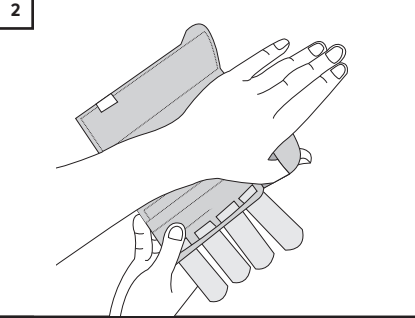
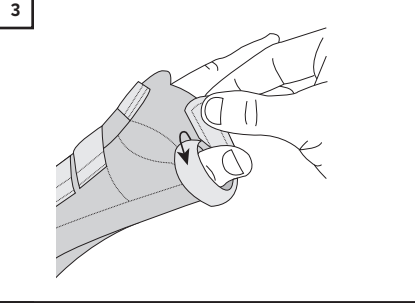
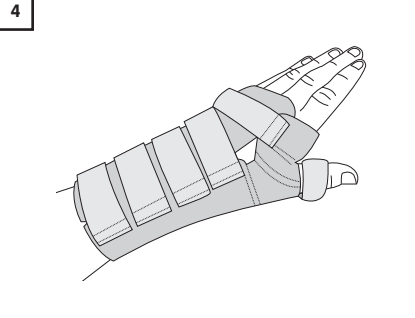
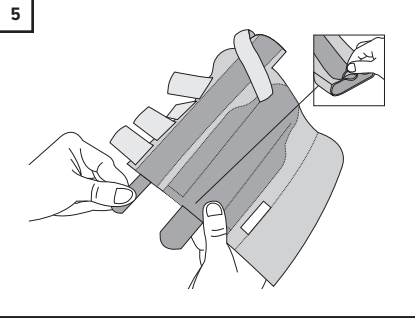
1	
2	
3	
4	
5	

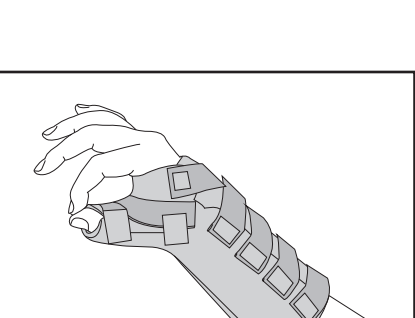
 MEDICAL DEVICE
 SINGLE PATIENT - MULTIPLE USE
 CAUTION

 Promedics Orthopaedics Ltd. Port Glasgow Industrial Estate, Gareloch Road, Port Glasgow, Scotland, UK, PA14 5XH Tel: +44 (0) 1475 746 400 Email: enquiries@promedics.co.uk www.promedics.co.uk
 EMERGO EUROPE Westervoortsewijk 60, 6827 AT Arnhem The Netherlands

			EMERGO EUROPE Westervoortsewijk 60, 6827 AT Arnhem The Netherlands
---	---	---	--

©2023 Promedics Orthopaedics Ltd. All Rights Reserved.

WTBIFUC revised 6/23

1	
2	
3	
4	
5	

Product Code BWTB has a unique one piece stay that is fixed in place. Shape as required and wash the brace with the stay in place.

ENGLISH

Product Description

A wrist thumb orthosis with malleable aluminium thumb stay to hold the thumb firmly in abduction, supporting the thumb MP joint and restricting movement at the CMC joint.

Usage Information (See diagram)

- Open hook / loop fastening and open brace out flat.
- Position thumb and hand in brace as shown.
- Close hook / loop fastenings.
- Close hook / loop fastening securely around thumb.
- Take care not to over tighten straps, re-adjust for comfort as required while ensuring orthosis is in correct position.
- The wrist and thumb stays can be removed, bent to the desired position (as required by the injury / condition) and re-inserted.

Indications for Use

- Provides wrist and thumb support for an array of injuries, conditions, pre- and post-operations such as:
 - Arthritis.
 - Sprains and strains.
 - Soft tissue injuries and conditions.
 - Instability.
 - Post fracture management.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps and remove metal splint.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.
- Before using brace again, replace metal splint ensuring splint is correct way round and splint is tucked under edge / 'lip' of pocket.

Product Family Composition

Aluminium, Chloroprene Rubber, Cotton, Elastane, Elastomer, Nylon, Polyamide, Polyester, Polypropylene, Polyurethane, PVC, SBR, Spandex, Steel, TPU, Urethane.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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