
WTW - Wrist Thumb Wrap

EN	WRIST/HAND ORTHOSIS
DE	HANDGELENK-/HANDORTHESE
FR	ORTHÈSE DU POIGNET/DE LA MAIN
ES	ORTESIS DE MUÑECA/MANO
IT	ORTESI PER POLSO/MANO
NL	POLS-HANDORTHESE
DK	HÅNDELED-/HÅNDORTOSE
FI	RANNE-KÄMMENORTOOSI
SE	HANDLEDS- OCH HANDORTOS
NO	ORTOSE, HÅNLEDD/HÅND
PL	ORTEZA NADGARSTKA/DŁONI
GR	ΝΑΡΘΗΚΑΣ ΚΑΡΠΟΥ/ΥΧΕΙΟΥ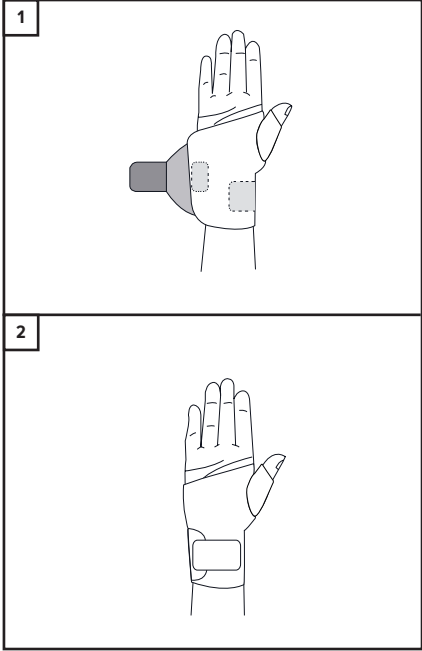

ENGLISH

Product Description

An orthosis for support and stabilisation of the wrist.

Usage Information (See diagram)

- Open hook / loop fastening and open brace out flat.
- Position hand in brace as shown.
- Close hook / loop fastening and re-adjust for comfort as required.
- Close hook / loop fastening and re-adjust for comfort as required.
- Ensure end of brace is not above palmar crease.

Indications for Use

Provides wrist support for an array of injuries, conditions, pre-and post-operations such as:

- Arthritis.
- Sprains and strains.
- Soft tissue injuries and conditions.
- Instability.
- Post fracture management.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the brace.

- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Elastane, Nylon, Polyamide, Polyester, SBR.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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